

HOW TO CLEAN, SANITIZE & STORE REUSABLE FACE MASKS



1 To remove mask take straps and pull forward. Or hold both ear loops. Unhook and gently lift to remove the mask.



2 Place mask in plastic bag until ready to wash. If the mask has a pocket that holds a removable non-woven filter, always remove the filter before washing the mask. Wash hands before and after removing mask.



3 Hand wash gently using normal washing detergent, with warm water (around 30°) for at least 60 seconds. Or wash mask daily in a washing machine. Use the warmest appropriate wash setting. You can also disinfect mask by soaking in a solution of hot water (70°) and detergent.



4 Dry masks completely. To dry, place in the dryer. Or use a hairdryer and keep it at least 6-8 inches away from the filter during the process. Or left to hang and air-dry under indirect sunlight.



5 If needed, iron cloth masks to help them keep their shape.



6 Store masks in a clean bag, paper bag or covered container before uses. You can also keep them in airtight bag. Store masks in cool and dry place or a place with moisture absorbers, such as bamboo charcoal or silica gel.